

Informative technical sheet 12-2017	
SPRAYGEN	
Ready-to-use alcoholic disinfectant.	
Medical Device C C 0476 II b Class BD CND D02010102 RDM 1530251	
Composition:	Benzalkonium chloride g 0,04
	Didecyldimethylammonium chloride g 0,06
	Chlorexidine digluconate g 0,05
	Isopropanol g 31,00
	Ethanol g 31,50
	Coformulants, denaturants and deionized water s.t. g 100
Aspect:	Clear colourless solution
Perfume:	Citrus
pH:	n.a.
<b>P</b>	
Biodegradability:	Over 90%
Toxicity:	Benzalkonium chloride acute oral LD <sub>50 rat</sub> : 445 mg/kg live weight
	Didecyldimethylammonium chloride acute oral LD <sub>50 rat</sub> : 238 mg/kg live weight
	Isopropanol acute oral LD <sub>50 rat</sub> : 5.045 mg/kg live weight
	Ethanol acute oral LD50 rat : 7.060 mg/kg live weight
	Chlorexidine acute oral LD <sub>50 rat</sub> : 1.260-1950 mg/kg live weight
Packaging:	1 liter bottle in a 12 pieces box including 2 spray triggers (further triggers may be supplied under request)
Characteristics and Properties:	The association of various active principles allows to obtain a large activity spectrum that includes gram+ and gram- bacteria, yeasts, mycetes, Mycobacterium tuberculosis and viruses (HIV-HBV-HCV, Adenovirus, Poliovirus). The high content of alcohol improves the effectiveness of active principles (synergic effect).
Use indications:	<ul> <li>Spray amply and thoroughly on surfaces and equipment.</li> <li>Leave it for at least 5 minutes according to the cleanness conditions of surfaces</li> <li>If necessary, dry.</li> </ul>
Applications:	• Do not rinse with water to keep a residual effect on surfaces. It is recommended to use the product on materials compatible with alcohol. Indicated for Dentist's surgeries, Analytical labs and Ambulatories
	VALIDITY: 36 months from the manufacturing date

**Confidential technical information for qualified personnel** 

pharma trade company srl - via torazza 115-121 20861 brugherio mb tel 039 2142186 fax 039 2872944 info@pharmatrade.it www.pharmatrade.it